medi USA, LP  
6481 Franz Warner Pkwy.  
Whitsett, NC 27377  
USA  
www.mediusa.com

medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth, Germany  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61-2 9890 8696  
F +61-2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T +32 11 24 25 60  
F +32 11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Brasil  
Rua Itapeva 538 cj. 42  
Bela Vista, São Paulo - SP  
CEP: 01332-000  
Brasil  
Tel: +55 (11) 3500 8000  
sac@medibrasil.com  
www.medibrasil.com

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medi. I feel better.

## circaid® non slip padding

The circaid non slip padding is worn directly against the skin, underneath any compression garment. Its tacky surface aids in preventing the garment from sliding.

### Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

### Material composition

circaid non slip padding:  
100% Polyurethane

### Washing instructions

The circaid non slip padding should be hand washed and air dried. Due to the nature of the material, debris will adhere to it; use tweezers to remove debris that does not wash out. Note that debris will decrease the effectiveness of the material over time.

- hand wash
- Do not bleach
- drip dry or line dry
- Do not iron
- Do not dry clean
- Do not wring

### Directions for creating - using garment

Trimming the circaid non slip padding (for use with a circaid garment)  
Step 1: Unfold the circaid non slip

padding and place it flat on a surface. Be sure to choose a clean surface as the material attracts debris.

Unroll the tab ends of the garment. Place the garment or portion of it (typically the thigh portion), flat on top of the circaid non slip padding with the black side of the material facing up.

Note: Do not let the hook tabs come in contact with the circaid non slip padding. The hook tabs may damage the padding and the fibers of the padding will decrease the effectiveness of the hook tabs.

Step 2: Using the legging as a template, cut a straight line through the circaid non slip padding. Use the outside edge of the tab ends as a guide.

Be sure not to cut the juxtafit garment, as this will void the manufacturer's warranty.

Step 3 If needed, trim the circaid non slip padding along the top and bottom edge of the garment to minimize excess material.

### Directions for creating - using circumference measurements

Trimming the circaid non slip padding (Using circumference measurements)  
Step 1: Measure the height of the desired coverage area on the limb.

Note: the maximum height of coverage area is 46 cm.

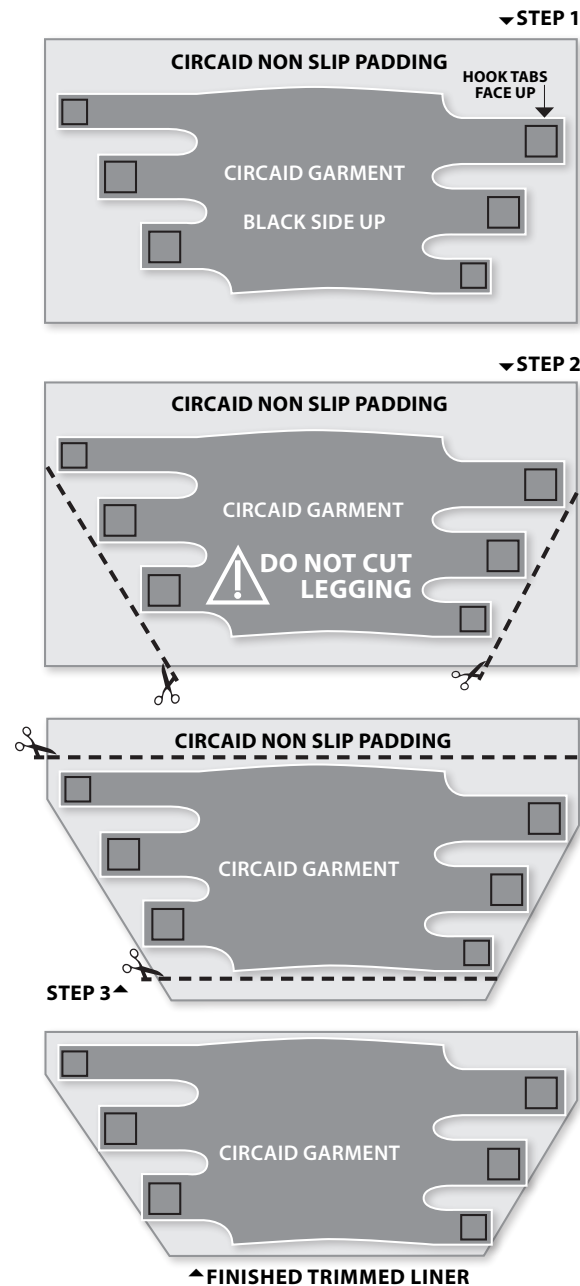
Step 2: Measure the largest and bottom circumference of the limb in the coverage area. Add 25 cm to each of these measurements to ensure your circaid non slip padding overlaps.

Step 3: Using these three measurements, mark and cut the padding per the drawing on the left cover.

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
United Kingdom  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Tankova Str. 8, office 35  
Business-center "Flora Park"  
Kiev 04112  
Ukraine  
T +380 44 591 1163  
F +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA, LP  
6481 Franz Warner Pkwy.  
Whitsett, NC 27377  
USA  
T +1 336 4 49 44 40  
F +1 888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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